Lehnen GmbH  
Kleinicher Straße 10  
D - 54472 Longkamp  
Tel. 06531/50057-0  
Fax 06531/50057-1022

Lehnen GmbH  
Kleinicher Straße 10  
D - 54472 Longkamp  
Tel. 06531/50057-0  
Fax 06531/50057-1022

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## Assembly instruction

Shower seats (wall mounted) NIL\_651A

You have chosen a Lehnen product. We are sure that the aid will give you more mobility and thus good service. Our products are manufactured according to strict quality guidelines.

Our instructions for mounting must be observed. The assembly should be carried out by a specialist company.

### Mounting material

The standard fastening material enclosed with each article consisting of:

Plastic multi-purpose dowel MZ 8  
Wood screw - raised countersunk head with Torx  
6 x 60mm

This mounting kit is suitable for mounting on the following substrates

Concrete B25  
Limes and brick Ks12  
Pumice solid V4  
Solid brick M20

For other substrates, tested fastening materials must be used. These must comply with the service load values listed below.

Breakaway load **600 N**

Group 12 = **Shower seat**

When selecting fasteners for other wall types as specified above, a specialist fastening company should be consulted. Check wall construction! In case of thick plaster layers, planking or wall structures, use correspondingly longer fastening material. The specifications of the anchor manufacturer must be observed.

### Mounting dimensions:

Shower seats are to be installed at a height of 460-480 mm finished floor / upper edge of seat surface.

A functional test must be carried out after assembly!

### Drilling method:

The holes are to be drilled with a drill bit d=8mm perpendicular to the substrate and at least 60mm deep. The drilling dust must be removed from the drill hole to ensure a tight fit of the anchor.

**Maximum load:** 150 kg

Stand: 07/23